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PLEASE READ CAREFULLY BEFORE USE. KEEP FOR REFERRAL PURPOSES.	EN
---	-----------

CREATION VC Ceramic Primer

Priming agent to treat the adhering surfaces of ceramic and composite prostheses. To promote a strong bond to resin-based materials as well as resin-based cements in case of luting procedures or direct composites in case of repairs.

For use only by a dental technician in the recommended indications.

INDICATIONS

- Primer for the adherent surfaces of indirect restorations made of ceramics (glass ceramics, zirconium dioxide and aluminium oxide).
- Intra- and extra-oral repair of fractured indirect restorations made of ceramics, hybrid ceramics and composites.

CONTRAINDICATIONS

Avoid use of this product in patients with known allergies to methacrylate monomer, methacrylate polymer or alcohol.

USE

1. Surface treatment before luting

- Treat the bonding surfaces of the prosthesis according to the respective manufacturer’s instructions and clean it using a suitable cleaning method (e.g. sandblasting with 110 µm aluminium oxide).
- Dispense one drop of CREATION VC Ceramic Primer into a dispensing dish.
- Apply a thin layer to the bonding surface using a micro-brush and blow excess with compressed air.
- Apply suitable resin-based luting cement to the treated surface. Observe the manufacturer’s instructions for use of the luting cement.

II. For intra-oral application

- Roughen the bonding surface of the ceramic or composite using a coarse diamond bur or a corundum grinding body.
- Treat the bonding surfaces of the restoration according to the respective manufacturer’s instructions for use and clean it using a suitable method. Rinse and dry with oil-free air.
- Dispense one drop of CREATION VC Ceramic Primer into a dispensing dish.
- Apply a thin layer to the fractured surface of the ceramic or composite prosthesis using a micro-brush and blow excess with compressed air.
- Continue the repair by using a light-curing adhesive (e.g. CREATION VC Composite Primer) and a light-curing composite (e.g. CREATION VC Flow Dentine) according to the respective manufacturer’s instructions for use.

Note: if the adherent surface to be treated is contaminated with saliva or blood, rinse with water and dry. Re-treat the surface according to the manufacturer’s instructions for use and re-apply CREATION VC Ceramic Primer according to the instructions above.

STORAGE

Recommended for optimal performance:

store at room temperature (4-25 °C). Keep away from direct sunlight.

SIZE OF PACKAGING

Bottle, 3 ml

CAUTION

- CREATION VC Ceramic Primer is flammable and volatile. Do not use near open flame. Keep away from sources of ignition. Keep away from direct sunlight. Only use in well ventilated rooms.
- In case of contact with oral tissues, immediately remove with a cotton pellet or a sponge. Rinse immediately after luting or repairing.
- In case of contact with skin, immediately remove with a swab. Thoroughly rinse with soap and water.
- In case of contact with eyes, immediately flush with water and seek medical attention.
- Replace the cap of the bottle immediately after use. If dew condensation is present on the bottle of CREATION VC Ceramic Primer, immediately wipe it off.
- Absolutely avoid swelling of the material.
- Do not mix with any other materials.
- Always wear personal protective equipment (PPE) such as gloves, face masks and safety eyewear.
- In rare cases the product may cause sensitivity in some people. If any such reactions are experienced, discontinue the use of the product and refer to a physician.

Some products referenced in this IFU may be classified as hazardous according to GHS. Always familiarize yourself with the safety data sheets available at: https://www.creation-willigeller.com/en/media-center/manuals-brochures/

CLEANING AND DISINFECTING

MULTI-USE DELIVERY SYSTEMS: to avoid cross-contamination between patients this device requires mid-level disinfection. Immediately after use inspect device and label for deterioration. Discard device if damaged. DO NOT IMMERSE. Thoroughly clean device to prevent drying and accumulation of contaminants. Disinfect with a mid-level registered healthcare-grade infection control product according to regional/national guidelines.

Undesired effects - Reporting: If you become aware of any kind of undesired effect, reaction or similar events experienced by use of this product, including those not listed in this instruction for use, please report them directly through the relevant vigilance system, by selecting the proper authority of your country accessible through the following link: https://ec.europa.eu/growth/sectors/medical-devices/contacts_en as well as to our internal vigilance system: vigilance@gc.dental. In this way you will contribute to improve the safety of this product.

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